



IF8000TM

APPLICATIONS

- * Symptomatic relief of chronic intractable post-traumatic and post-surgical pain (IF-mode)
- * Relaxation of muscle spasms
- * Prevention or retardation of disuse atrophy
- * Increasing local blood circulation
- * Muscle re-education
- * Immediate post-surgical stimulation of calf muscles to prevent deep venous thrombosis
- * Maintaining or increasing range of motion

INTERFERENTIAL THERAPY

Unique Features

- * The IF8000 offers two-channel interferential operation with four electrodes, as well as a two electrode configuration.
- * The IF8000 has five frequency shift options, allowing you to program the unit for specialized rehabilitation needs.
 - C Continuous mode. Beat frequency remains constant during the entire treatment period.
 - 1/1 Abrupt. Beat frequency shifts in one second durations from 30% below to 60% above the set frequency. (Thus at frequency of 10 Hz at 1/1, the net delivered beat frequency would modulate from 7 Hz to 16 Hz every other second).
 - 1/1 Ramped. Beat frequency ramps in one second durations from 30% below to 60% above the set frequency.
 - 6/6 Abrupt. Beat frequency shifts in six second durations from 30% below to 60% above the set frequency.
 - 6/6 Ramped. Beat frequency ramps from 30% below to 60% above the set frequency in 6 second intervals, with a slow, up and down ramping transition.
- * The IF8000 has a constant current output that makes it safer and more comfortable than most other available devices.
- * The IF8000 automatically shuts off after 20 minutes of treatment or other preset time.
- * The IF8000 is portable, wearable, and easy to use.
- * The IF8000 is covered by a three (3) year limited warranty.
- * The IF8000 is controlled by a microprocessor that allows for advanced programming with many safety features.
- * The IF8000 has a digital display.
- * The IF8000 records compliance data, such as usage time and number of times used.
- * Unique protection against reverse battery polarity - the IF8000 will work with either polarity.

SPECIFICATIONS:

- * Carrier Frequency: 4000 Hz nominal, fixed frequency.
- * Adjustable Frequency: Continuous: 4001-4150 Hz nominal, Frequency Shift Mode: 4001-4240 Hz nominal.
- * Interference Frequency: 1-150 Hz (Continuous mode), 1-242 Hz (Frequency Shift mode).
- * Output Configuration: Interferential (4 electrode) and premixed interferential (2 electrode).
- * Frequency Shift Options: Continuous, 1/1 ramped, 1/1 abrupt, 6/6 abrupt, 6/6 ramped
- * Waveforms: Symmetric biphasic square with zero net DC.
- * Output Current: 0-50 mA peak adjustable. (0-50 V peak into 1000 Ω)
- * Pulse width: 125 usec.
- * Power Source: One (1) 9V battery or AC adapter

All electrical specification are +/- 10% into real 500 ohm resistive load.

ZYNEX
MEDICAL
9990 Park Meadows Dr.
Lone Tree, CO 80124
Phone (866) 503-8921
Fax (800) 495-6695
www.zynexmed.com